

Shield against infection.

Ensure your P2/N95 face mask is giving you the protection you need.

Nosedip should be moulded around the nose

Tip

Wait until you're in clean air to remove your mask.

**Upper strap should be positioned on the crown of the head.
Strap should not be twisted.**

Tip

Check you have the right size of mask if you're having issues with the seal.

Respirator should be correctly positioned on your face and head.

Tip

Face masks should be fitted and checked before entering an unsafe environment.

Make sure the panels are fully unfolded

**Lower strap should be positioned below the ears.
Strap should not be twisted**

Conduct a fit check

Check the seal of the mask by gently inhaling. If the mask is not drawn in towards your face or air leaks around the face seal, readjust the mask.

For everything your workplace needs to work safely, shop our range at winc.com.au or ask to speak to a Winc Health, Hygiene and Safety Specialist.